
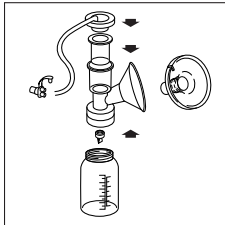
EN Elite Breast Pump

ES Tiraleche Elite


CA Tire-lait Elite

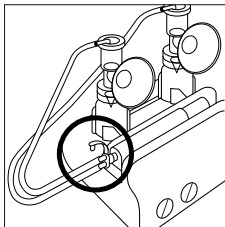
1

2

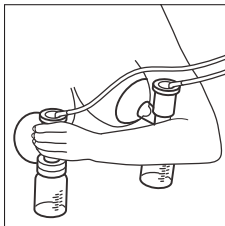3

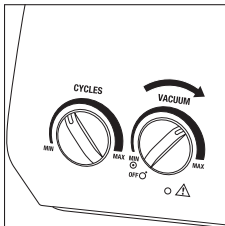4

EN Operating the Elite Pump

IMPORTANT NOTE: ONLY USE THE AMEDA HYGIENIKIT MILK COLLECTION SYSTEM.

1. Assemble the Ameda HygieniKit Milk Collection System as per the instructions included with the product.
2. Insert the tubing adapter on the HygieniKit Milk Collection System into the piston cylinder opening.
3. Center your nipple(s) into the breast flange(s) of the HygieniKit Milk Collection System.

Starting the Elite Pump

4. Turn on the vacuum dial. Increase the suction by turning it to the right. Start at the highest comfortable setting.

NOTE: High suction does not necessarily produce more milk.
(Continue to reverse side for additional instructions.)

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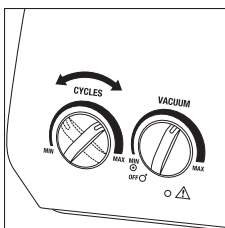
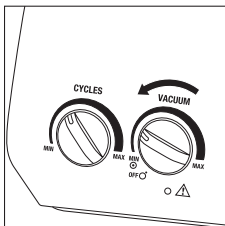
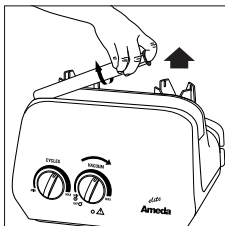
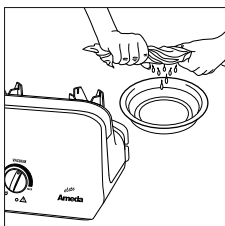
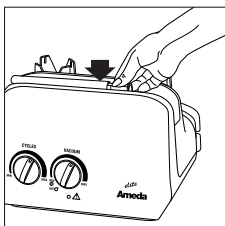
9

EN *Setting the Cycle Speed*

5. Turn the cycle dial all the way to the right to increase speed and help trigger a milk release (let-down). Once your milk begins to flow, turn the dial back to the left.

As your milk begins to slow down, repeat step 5 to trigger more let downs.

6. When finished pumping, turn the vacuum dial all the way to the left to turn the pump off.

Cleaning the Pump

7. Remove piston cylinder by placing index finger into the tube, lifting upwards and turning right to release from attachment bars.

8. **For end user:** After each pumping, wipe pump with cloth dampened with warm, soapy water.

For hospitals and rental stations: Clean between users. Spray with recommended disinfectant, such as CaviCide. Wipe with soft cloth. Do not use products containing phenol. Do not use bleach on pump.

9. Place piston cylinder back onto attachment bars and press gently into place with thumb.

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