Ameda Pearl Electric Breast Pump

QUICK START CARD

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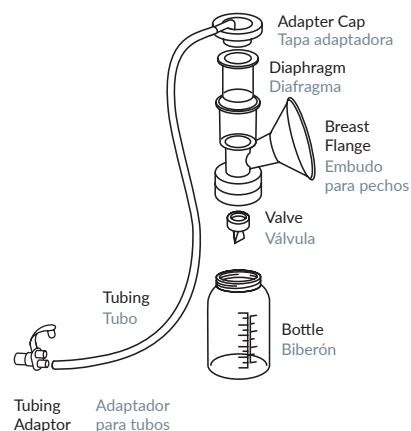
GETTING STARTED: Before first use, wash hands thoroughly with soap and water before touching breast pump, kit and breasts.

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1

- If kit is sterile, use preassembled kit immediately.
- If not sterile, clean, sanitize and assemble the kit (follow cleaning instructions on the back of this card).

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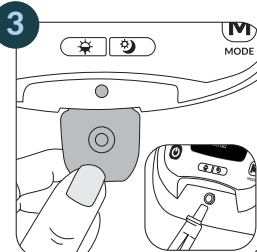
2

- First connect one end of the power adapter to the back of the pump, then the other end to an electrical outlet.

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
3

- Insert tubing adapter firmly into port under gray flap on front of pump.
- If not double pumping, REMOVE ONE TUBE and insert the attached plug into the unused port of the adapter.

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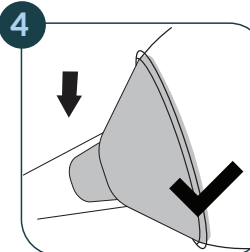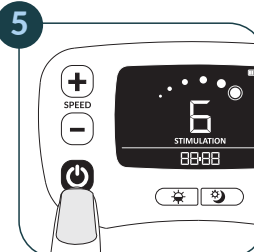
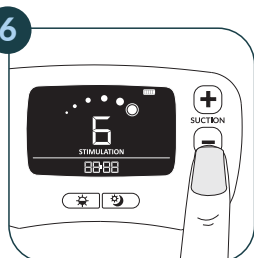
6


Adjust Stimulation Suction

- Adjust the Stimulation Suction setting gradually to your **preferred comfort** level (maximum 6).

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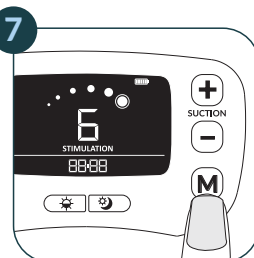7


Change Mode

- Once your milk starts to flow using the STIMULATION MODE, press the Mode (M) button to switch into Expression Mode.

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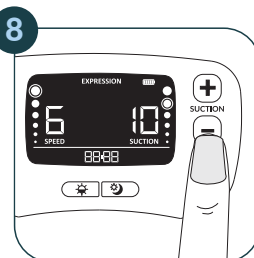8


Adjust Expression Suction Level

- Adjust the Expression Suction setting gradually to your **preferred comfort** level. (maximum 12).

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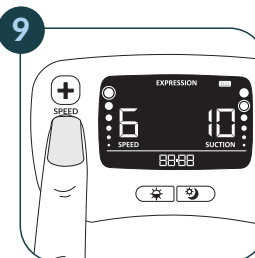9


Adjust Speed

- Adjust the speed in Expression Mode to your **preferred comfort** level (maximum 6).
- Once your milk stops flowing freely while in Expression Mode, press the Mode button to go back to Stimulation Mode to induce a second let-down if desired.

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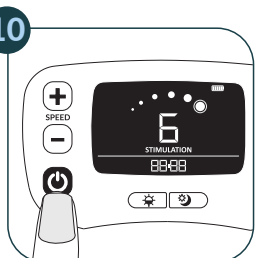10

End Pumping Session

- When finished pumping, press power button to turn off.

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Cleaning the Ameda Pearl Electric Breast Pump

For At-Home/End Users:

After each pumping, turn pump OFF.
Wipe with clean, damp cloth.

For Hospital/Rental Stations:

Turn pump OFF. Clean between users wearing disposable gloves. Use a hospital approved antibacterial cleaner such as CaviCide®, CaviWipes®, or PDI® Sani-Cloth® AF3.

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
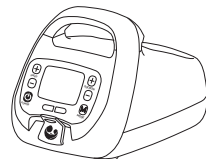
Cleaning the Ameda HygieniKit

IMPORTANT: Do Not Wash Tubing Adapter, Silicone Tubing, Adapter Cap. They do not contact milk during pumping.

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
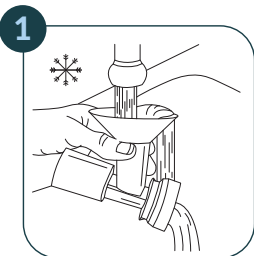
RINSE any milk from kit parts with cool water.

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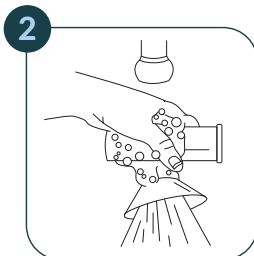
Hand WASH parts with warm, soapy water. Do not insert anything into valve or diaphragm while cleaning.

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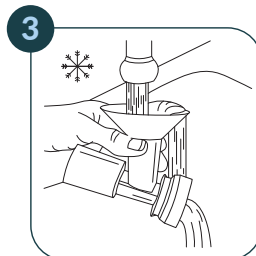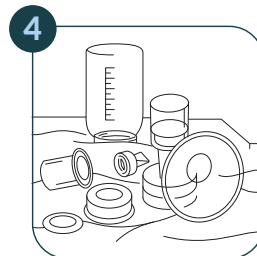Air-dry thoroughly on a clean surface.

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Breast Pump Valve and Diaphragm Check

Inspect valves and diaphragms for tears or holes. Replace any damaged valves and diaphragms prior to pumping. Replace valves every 8-12 weeks and diaphragms every 4-6 weeks.

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SCAN ME

Consult Instructions for Use
Ameda Pearl Electric Breast Pump
and HygieniKit

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FIND YOUR BEST FIT

A good breast flange fit is essential for comfort and pumping efficiency.

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Good Fit

During pumping, nipple moves freely in the breast flange tunnel. You see space around the nipple. Not much areola is drawn into the tunnel.

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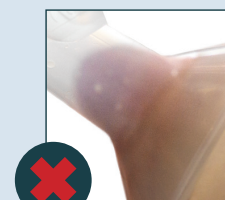
Too Small

During pumping, nipple rubs against the sides of the tunnel.

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Too Large

During pumping, areola is drawn into the flange and may rub against the side of the tunnel.

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